

multi-CLIN™ Drug Screen Test Device

Please read the instructions carefully before performing the test.

1. Add 100µL of urine (3 drops using pipette) to EACH sample well.

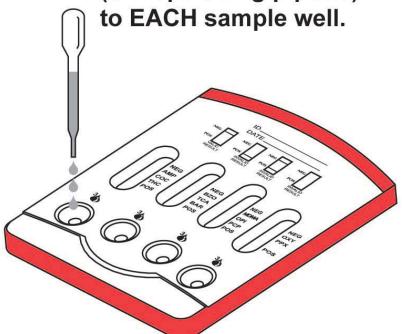


2. Wait 5 minutes to read results.

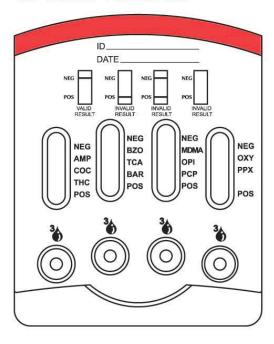



3. Confirm validity of results by checking NEG and POS Control Regions.

There should be NO line in the Positive Control Region (POS) and a visible line in the Negative Control Region (NEG).

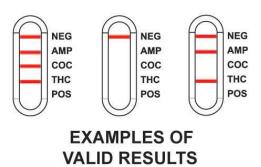


4. Interpret Results at 5 minutes.

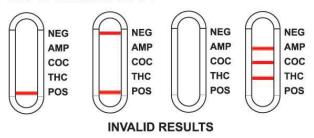

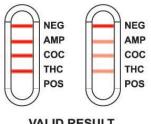

VALID RESULT, NEGATIVE for ALL ANALYTES SHOWN

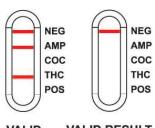

VALID RESULT, POSITIVE for COC VALID RESULT, POSITIVE for ALL ANALYTES SHOWN