



# Instructions

Learn More About  
iScreen One Step Drug Screening Kits

 EmployeeDrugTestingAce.com  
Trusted Proven Secure

[www.DrugTestingAce.com](http://www.DrugTestingAce.com)

[www.EDTAce.com](http://www.EDTAce.com)

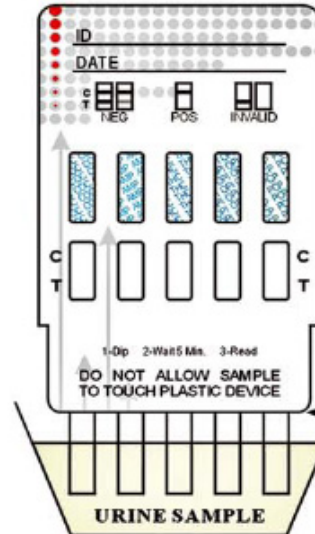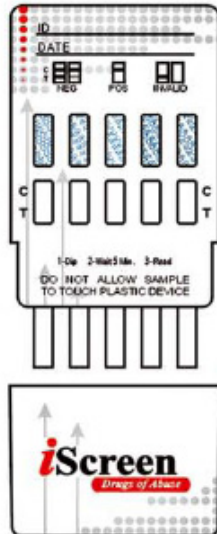

This illustration shows  
a 5-Drug Test Card

Do not allow specimen to  
touch the device housing

1. Remove iScreen™ test from foil pouch at room temperature. Take off the cap revealing the testing strips.
2. Immerse test vertically in the urine sample for at least 10 - 15 seconds. Do not allow the specimen to touch the device housing.
3. Read at five (5) minutes.  
Results stable up to four (4) hours.



## Interpretation of Results:



**Negative**

(2 Lines - Any indication of a line in both test and control fields is to be interpreted as a NEGATIVE)



**Non-Negative**

(1 Line)



**Invalid**

(No line(s) or no control line(s))