




iScreen™ One Step Drug Test Interpretation Quiz

Learn More About
iScreen One Step Drug Screening Kits

 **Employee Drug Testing Ace.com**
Trusted Proven Secure

www.DrugTestingAce.com www.EDTAce.com

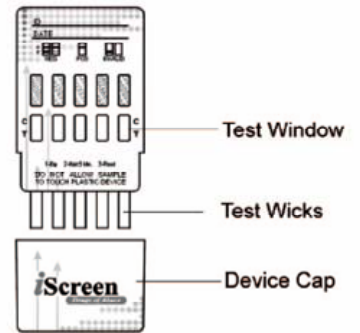

I. MULTIPLE CHOICE

- When Collecting Specimen in a cup, what should the temperature range be?
 - 90-100° Celsius
 - 85-95° Fahrenheit
 - 90-100° Fahrenheit**
 - 85-95° Celsius
- The absence of a line next to a T or number line indicates a _____ for the particular drug.
 - Non negative**
 - Negative
 - Invalid
 - All of the above
- When administering the test, immerse the test strips vertically in the urine sample for at least:
 - 30-45 seconds
 - 1 minute**
 - 10-15 seconds
 - None of the above
- Negative results may be interpreted as soon as all the lines are visible. Non Negative results may be interpreted for as long as:
 - 10 minutes
 - 1 minute**
 - 3 minutes
 - 5 minutes

II. MATCHING (Match the appropriate abbreviation to the drug name)

- | | | |
|---------------------|----------------------|---------|
| 5. Marijuana | _____ F _____ | a) COC |
| 6. Amphetamine | _____ D _____ | b) MDMA |
| 7. Cocaine | _____ A _____ | c) mAMP |
| 8. Opiates/Morphine | _____ E _____ | d) AMP |
| 9. Ecstasy | _____ B _____ | e) OPI |
| 10. Methamphetamine | _____ C _____ | f) THC |

III. IDENTIFY THE RESULTS AS EITHER: (A) NON NEGATIVE, (B) NEGATIVE, OR, (C) INVALID



— **B** —



— **A** —



— **C** —

Brought to You by Employee Drug Testing Ace

Buying discounted, accurate, high quality employment drug testing kits and services is affordable, fast, easy, safe and secure with **Employee Drug Testing Ace**. You can buy employment, pre-employment and employee drug and alcohol testing kits, services and supplies tailored for employers, TPAs, nursing homes, hospitals, manufacturing, moving companies, Department of Transportation DOT, local and Federal Government agencies, auto dealerships, construction, DOD, the work place, employment agencies and more.

Our easy to use [employee drug testing](#) kits are DOT Approved, CLIA Waived, FDA 510k Cleared and SAMHSA Compliant. We provide both onsite employment drug screening and offsite employee drug screening services along with SAMHSA certified confirmation and medical review officer (MRO) services. All [employment drug screening](#) kits are on sale, discounted, and have [bulk / volume discounts](#).

- **Pay No Taxes in the USA.**, except California.
- **100% Free UPS Standard Shipping** in the Continental U.S.A. for *All* Products.
- **100% Free FedEx Overnight Shipping** in the Continental U.S.A. for *All* Specimens sent in as part of our SAMHSA Certified Confirmation Drug Testing Service.
- Onsite Third Party Administered (TPA) Employee Drug and Alcohol Collection Services.
- Over 12,000 Offsite Employee Drug and Alcohol Testing Locations Nationwide.
- The most complete [Employment Drug Testing Resource Center](#) including: information, official documents, work place drug testing information, forms, work accident info, charts, videos, post accident drug testing info, training materials, contact lists, definitions and more!

We sell affordable and guaranteed [employee drug screening](#) kits and services available at the best prices. Our prices may be cheap but our drug testing kits are of the highest quality and accuracy. If you are in the market for discounted high quality pre-employment, employee, or [employment drug testing](#) and screening kits and services, visit us today. You will be happy you did!